



Clinical Overview



Features and Benefits of the Cleanlet 300 Electronic Bidet

Overview

The Cleanlet 300 series bidet attaches to the existing toilet pan in place of the seat and lid and draws cold water from the cistern tap. When the Cleanlet bidet is plugged into a 230V 3-pin socket the water will be heated in the small tank inside the unit. The warm water is then piped to the wash nozzle when the “wash” button on the control panel is pressed. The cistern flush is unaffected by the installation of the Cleanlet.

Features

- Ease of use
- Warm water wash
- Variable water pressures
- Warm air dry
- Variable air temperatures
- Heated seat
- Manual or Remote control models
- Soft Close seat & lid

Benefits

- Assists people who cannot manage their personal hygiene by themselves
- Retains independence and dignity and make life easier
- Assists in controlling urinary tract infections
- Assists in the relief of mild constipation
- Eliminates the use of toilet paper for those loose bowels or skin problems and find wiping uncomfortable
- Hygiene is the primary roll of the Cleanlet Bidet

Anal Cleansing

This nozzle has three jets providing cleaning of the anal area. The water pressure can be adjusted to meet personal requirements. There is the choice of local washing or by pressing the wash a second time the wash area is elongated. The water temperature can be preset.

Constipation

Constipation is commonly encountered in older adults, as they are more susceptible to chronic physiologic and pathologic change that can predispose them to constipation.

Clients with mild constipation have found that by using the “turbo” wash and allowing the warm/hot water to spray around the anal area a bowel movement more than likely results.

Diarrhoea

With the choice of a warm or cold water wash to soothe the area, Cleanlet can make life much more comfortable.

In a case where a male client was on medication the consequences, in this instance, was a 3-day bout of Diarrhoea every three to four weeks. Although the Cleanlet Bidet did not prevent the complaint the soothing wash and the air-dry that eliminated wiping made life more bearable.

Vaginal Cleansing

This wash function has six jets of water. The water pressure is milder with increased volume of water. There is evidence that the aging urinary tract is associated with a decrease in cell-mediated immunity, and as a result the older patient is more susceptible to urinary tract infections (UTIs). Other antibacterial factors also decrease with aging.

One of our clients has had many years of infections and in her words was “either going on or coming off antibiotics most of the time.” In the nine years she has had a Cleanlet Bidet, she has not experienced another UTI.

Overweight People

The Cleanlet Bidet can accommodate persons up to 150 kg. However, it is not the physical weight that can damage the Cleanlet but the impact force of “plonking” onto the seat. If the lowering of the body onto the Cleanlet seat can be controlled, the benefits of a water wash and the air dry can be life changing.

The need for wiping is eliminated.

To further assist, it is possible to fit a spacer under the Cleanlet bidet to give added height.

Also available is a specially designed Bariatric frame consisting of hand rails to give support and added width.



*Scan here to see the bidet in action as a Clinical Overview
or go to www.clinical.cleanlet.co.nz*